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Mindray Beneheart C1A / C2

Kontrolle Gerätestatus

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
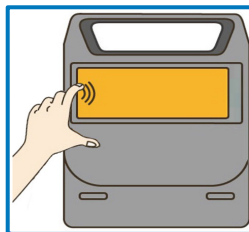
Abb. 3

Abb. 4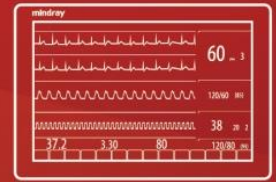

Country	Switzerland
Contact	Procamed AG
Project	Manual addendum
Device	BeneHeart C-Series

This memo is intended to explain chapter 8.2 of the BeneHeart C series manual.

Regarding "maintenance" under chapter 8.2. Mindray C-Series AED are maintenance free and do not require any special maintenance. However, the device needs 'inspection'. A visual inspection of the battery and pads. We recommend to do this inspection at least once per month.

8.2 Maintenance Safety Information

WARNING

- Failure for the responsible institution employing this equipment to implement a satisfactory maintenance schedule may cause undue equipment failure and delayed analysis.
- No modification of this equipment is allowed.
- This equipment contains no user serviceable parts.
- The safety checks or maintenance involving any disassembly of the equipment should be performed by professional servicing personnel. Otherwise, undue equipment failure and possible health hazards could result.
- The service personnel must be properly qualified and thoroughly familiar with the operation of the equipment.

The check and the replacement of the pads and battery can be replaced by anyone. No special training is needed. Pads and battery are not parts but disposables.

For any related questions, feel free to reach out to me.

Kind Regards,

Michel Rijnberk

Patient Monitoring Life Support
Business Development Manager AED - Europe

e-mail: michel.rijnberk@mindray.com

mobile: +31 645868895

G Inspection Record

Current Date: Month/Year: ____/____
Page: ____ of ____ in the corresponding file

Daily Checklist					
Inspected Date	Status Indicator (Green/Red)	Inspected by	Inspection Date	Status Indicator (Green/Red)	Inspected by
1.	in green / in red		17.	in green / in red	
2.	in green / in red		18.	in green / in red	
3.	in green / in red		19.	in green / in red	
4.	in green / in red		20.	in green / in red	
5.	in green / in red		21.	in green / in red	
6.	in green / in red		22.	in green / in red	
7.	in green / in red		23.	in green / in red	
8.	in green / in red		24.	in green / in red	
9.	in green / in red		25.	in green / in red	
10.	in green / in red		26.	in green / in red	
11.	in green / in red		27.	in green / in red	
12.	in green / in red		28.	in green / in red	
13.	in green / in red		29.	in green / in red	
14.	in green / in red		30.	in green / in red	
15.	in green / in red		31.	in green / in red	

Monthly Checklist:
Expiration date of electrode pads: _____