

April 8, 2020

Dear Valued Customer:

Thank you for your interest in the ZOLL 731 and Z Vent Ventilators. Currently, the cleaning instructions for these products per the Operators Guide is:

*Clean the device's housing and hose connections with a damp, soapy cloth. For general decontamination and cleaning, apply a 10% bleach solution with a damp cloth. Do not clean the device with abrasives or chlorinated hydrocarbon cleansers, which damage the housing and interface lens. After cleaning, thoroughly dry the device with a lint-free cloth. Make sure that all exposed surfaces are cleaned and dried.*

With risks associated with extremely contagious pathogens and the desire to standardize recommended cleaning agents across ZOLL devices, ZOLL is now suggesting the following additional cleaning agents not listed in the Operators Guide:

- Clinell® Universal Wipes
- Ethanol Disinfection with detergent
- Super Sani-Cloth® Wipes
- Sani-Cloth Plus® Wipes
- Oxivir® TB and Oxivir® 1 Wipes
- CaviWipes® XL Wipes
- Clorox Healthcare Hydrogen Peroxide Wipes
- 70% Isopropyl Alcohol (IPA) Wipes
- Sodium Hypochlorite (Bleach) Solution or Wipes (up to 10,000 ppm)
- Hydrogen Peroxide Solution
- Wex-Cide® 128 Solution
- Coverage Spray HB Plus
- Warm Water and Soap

Refer to the device Operator's Guide for guidance on cleaning the device as well as for detailed operating instructions. Local procedures and instructions provided by the cleaning agent manufacturer should always be followed for any cleaning or disinfection procedures. Summaries of cleaning instructions and additional information on infection diseases can be found here:

<https://www.zoll.com/resources/covid19/index>.

If there are any specific inquiries about the device, please contact your local ZOLL Technical service representative.

Best regards,



Fred Newey  
Director, Global Product Management  
ZOLL Resuscitation