ZOLL AED 3[®]

Training Power Adapter

ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA USA
01824-4105

ZOLL International Holding B.V.
Newtonweg 18
6662 PV ELST
The Netherlands
Tel.: +31 (0) 481 366410

Copyright © 2021 ZOLL Medical Corporation. All rights reserved. ZOLL and ZOLL AED 3 are registered trademarks of ZOLL Medical Corporation in the United States and/or other countries.

EN

Overview

The ZOLL AED 3[®] Training Power Adapter allows you to connect a ZOLL AED 3 to an AC power source for use in a non-clinical environment.

The ZOLL AED 3 Training Power Adapter is intended for use during training or device configuration only.

Warning

The ZOLL AED 3 Training Power Adapter is not intended for patient use in a clinical environment.

The ZOLL AED 3 readiness indicator will be cleared if the AED is powered off with the Training Power Adapter installed.

The ZOLL AED 3 performs a 2J self-test when the Training Power Adapter is installed and when the AED is powered on. No energy is delivered when using the Training Power Adapter.

DO NOT store the Training Power Adapter in a ZOLL AED 3 or near devices intended for clinical use.

When using the ZOLL AED 3 for training purposes, keep a ZOLL AED 3 Battery Pack and a clinical electrode nearby to be used in the event of a medical emergency.

The ZOLL AED 3 Training Power Adapter is not intended for use near intentional transmitters of radio energy, such as radar installation or radio transmitters.

This device has NO SERVICEABLE PARTS. DO NOT disassemble the Training Power Adapter. Refer all servicing to qualified personnel.

Caution

This device is ONLY COMPATIBLE WITH ZOLL AED 3 SOFTWARE RELEASE 6 AND HIGHER. ZOLL AED 3 devices running software release 5 and lower will automatically shut down within 10 seconds when connected to a Training Power Adapter.

To prevent an electric shock when plugged into AC power, check for damage to the Training Power Adapter and power supply cord/plug prior to plugging into the ZOLL AED 3. Position the power cord for an accessible disconnect from AC mains power. Use only the provided AC power supply, ZOLL part number 0301-00013[X], with the ZOLL AED 3 Training Power Adapter.

DO NOT IMMERSE OR SPRAY TO CLEAN. After use, clean the Training Power Adapter with a soft, damp cloth using either 90% isopropyl alcohol, or soap and water.

The following symbols may be used on this equipment:

Symbol	Description
	Conformité Européenne Complies with medical device directive 93/42/EEC.
	Manufacturer.
	Authorized representative in the European Community.
	Serial number.
	Catalog number.
	Refer to manual.
	Return to a collection site intended for waste electrical and electronic equipment (WEEE). Do not dispose of in unsorted trash.
	Keep away from open flame and high heat.
	Do not open, disassemble, or intentionally damage.
	Do not crush.
	Not for patient use.
	Direct current.

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