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Mindray Beneheart C1A / C2

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
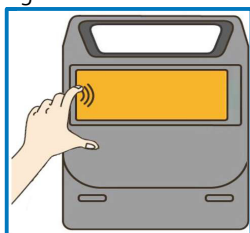
Fig. 3

Fig. 4

Europe - PMLS/defibrillator | Product Support Memo

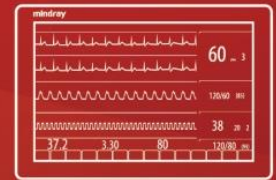Country	Switzerland
Contact	Procamed AG
Project	Manual addendum
Device	BeneHeart C-Series

This memo is intended to explain chapter 8.2 of the BeneHeart C series manual.

Regarding "maintenance" under chapter 8.2. Mindray C-Series AED are maintenance free and do not require any special maintenance. However, the device needs 'inspection'. A visual inspection of the battery and pads. We recommend to do this inspection at least once per month.

8.2 Maintenance Safety Information

WARNING

- Failure for the responsible institution employing this equipment to implement a satisfactory maintenance schedule may cause undue equipment failure and delayed analysis.
- No modification of this equipment is allowed.
- This equipment contains no user serviceable parts.
- The safety checks or maintenance involving any disassembly of the equipment should be performed by professional servicing personnel. Otherwise, undue equipment failure and possible health hazards could result.
- The service personnel must be properly qualified and thoroughly familiar with the operation of the equipment.

The check and the replacement of the pads and battery can be replaced by anyone. No special training is needed. Pads and battery are not parts but disposables.

For any related questions, feel free to reach out to me.

Kind Regards,

Michel Rijnberk

Patient Monitoring Life Support
Business Development Manager AED - Europe

e-mail: michel.rijnberk@mindray.com

mobile: +31 645868895

G Inspection Record

Current Date Month/Year: ____ / ____
Please use "X" in the corresponding box

Daily Checklist			
Inspected Date	Status Indicator Battery	Inspected By	Inspection Date
1.	☐ in green ☐ in red		17.
2.	☐ in green ☐ in red		18.
3.	☐ in green ☐ in red		19.
4.	☐ in green ☐ in red		20.
5.	☐ in green ☐ in red		21.
6.	☐ in green ☐ in red		22.
7.	☐ in green ☐ in red		23.
8.	☐ in green ☐ in red		24.
9.	☐ in green ☐ in red		25.
10.	☐ in green ☐ in red		26.
11.	☐ in green ☐ in red		27.
12.	☐ in green ☐ in red		28.
13.	☐ in green ☐ in red		29.
14.	☐ in green ☐ in red		30.
15.	☐ in green ☐ in red		31.

Monthly Checklist
Expiration date of electrode pads: _____